

# 3 Ply Disposable Mask



## Features and Benefits

- Class one surgical mask that is certified by the FDA. The three-ply construction offers a bacterial filter to protect the user.
- Suitable for most faces. This mask is close fitting and adapts to the contour of most faces.
- Fatigue Resistant. The straps are designed to eliminate ear pain; even after long durations of wear.
- Hypoallergenic. This mask is formulated to mitigate skin irritation.
- Adjustable Nose Clip to seal the nose bridge, preventing harmful airborne particulate matters and aerosols from penetration, and avoiding foggy glasses.
- Disposable. No need to worry if the mask has been sanitized after a previous use as they are disposable and designed not to be reused.
- Perfect for use in general day to day activities such as going to the supermarket or use in restaurants.

## When to Use a Mask

- The CDC recommends wearing cloth face coverings in public settings where other social distancing measures are difficult to maintain.
- As a best practice, the FDA recommends that all employees should wear a face mask during working hours.
- Wear a mask if you are displaying any signs of illness such as coughing or sneezing.

## How to wear disposable masks to protect against coronavirus


- Before putting on a mask, clean hands with alcohol-based hand rub or soap and water.
- Cover mouth and nose with mask and make sure there are no gaps between your face and the mask.
- Avoid touching the mask while using it; if you do, clean your hands with alcohol-based hand rub or soap and water.
- Replace the mask with a new one as soon as it is damp and do not reuse single-use masks.
- To remove the mask: remove it from behind (do not touch the front of mask); discard immediately in a closed bin; clean hands with alcohol-based hand rub or soap and water.



1. Clean your hands with soap and water or hand sanitizer




2. Hold the mask with the colored side away from you.

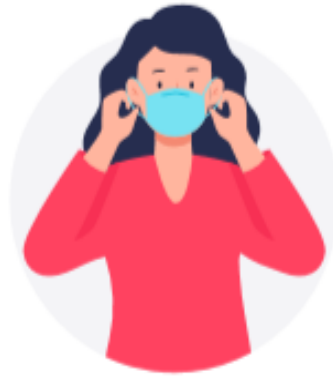

3. Hold the mask by the ear loops and place a loop around each ear.



4. Mold the stiff edge to the shape of your nose.

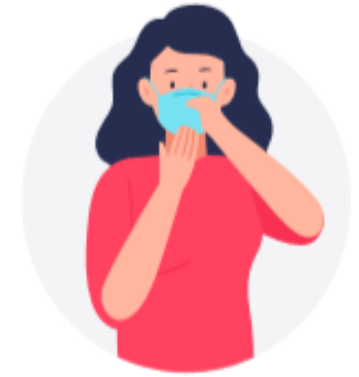

5. Pull the bottom of the mask over your mouth and chin.



### Available (Nationally)

**MASK FACE MEDICAL  
BFE 98% 3PLY**

**SUPC:** 5479538

**PACK/SIZE:** 40/50CT

**BRAND:** GIA

**MASK FACE PROTECT  
BFE 95% 3PLY**

**SUPC:** 5480047

**PACK/SIZE:** 40/50CT

**BRAND:** GALLIMORE

### Available (Regionally)

**MASK FACE PROTECT  
BFE 95% 3PLY**

**SUPC:** 7130077

**PACK/SIZE:** 80/50CT

**BRAND:** SAFETYX

**MASK FACE PROTECT  
BFE  
95% 3PLY**

**SUPC:** 5482748

**PACK/SIZE:** 40/50CT

**BRAND:** RAZOR

**MASK FACE PROTECT  
BFE 95% 3PLY**

**SUPC:** 7126692

**PACK/SIZE:** 1/50CT

**BRAND:** OPCOEMP